



External Battery Charger

User Manual

Thank you for choosing our products.
Please read this user manual carefully before use.

Product Property

- 1.Name: External battery charger
- 2.Model: FC-101
- 3.Dimensions: 85.5mm x 79.8mm x 34.4mm

Product Characteristic

Unauthorized charging of other batteries may cause electric shock or burst.

Electric Parameters

- 1.Input: DC20 V-100 VA
- 2.Output: DC16.8 V-5A MAX

1

Ambient Condition

Suggested using environment: Indoor use only;
temperature from 41°F (5°C) to 86°F (30°C)

Safety Requirements

Compliant with safety standards

Operating Instructions

- 1.Slide the battery along the track until you hear a click to confirm that the battery buckle has locked in place.
- 2.Plug in to start battery charging.

Cautions

- The new rechargeable battery can achieve best performance after being used for 2 or 3 times.

2

- If the rechargeable battery is stored for above one week, part of its power will run off. Therefore the battery should be recharged before use.
- It is normal for the battery to go warm slightly while charging. After the battery is fully charged, its temperature will gradually return to room temperature.
- To achieve best charging performance, please use the charger in a well-ventilated room.
- Children can use this charger only under the supervision of adults.
- Use in a proper way and ensure a correct slide-in.
- Disconnect the external power supply when charging completes.
- For Weoxy oxygen concentrators only. Use for other purposes is forbidden.

3

Warnings

- Unauthorized charging of other batteries may cause electric shock or burst, and even physical injury or property loss.
- Improper use of charger may cause electric shock.
- Indoor use only and keep from heat and moisture.
- Do not disassemble, assemble the charger or use it for other purposes.
- Please keep the charger out of the reach of children.



For indoor use
only 32°F - 113°F



No disassembly



Proper disposal



4

Warranty Card

Product name: External battery charger
Model:FC-101

Warranty Policy

- 1.Within 7 days from the second day of your receipt, this warranty only applies to product failures caused by manufacturing defects or material issues under normal use conditions. After confirmation by the Weoxy After-sales Service Center, you can enjoy free sales return or exchange services.
- 2.Between 8 and 15 days from the second day of your receipt, this warranty only applies to product failures caused by manufacturing defects or material issues under normal use conditions. After confirmation

5

by the product After-sales Service Center, you can enjoy free sales return or exchange services.

- 3.Within 12 months from the second day of your receipt, this warranty only applies to product failures caused by manufacturing defects or material issues under normal use conditions. After confirmation by the product After-sales Service Center, you can enjoy free sales return or exchange services.

Non-warranty Policy

- 1.Unauthorized repairs, misuse, collision, negligence, abuse, liquid ingress, accidents, modifications, incorrect use, or tearing or altering of labels, anti-tamper marks;
- 2.Product performance failures caused by human factors.

6

WeOxy Health, LLC
110 Cedar St, Suite 105, Wellesley, MA 02481 USA

7